



BOVINE & EQUINE PRODUCT CATALOGUE & OPERATION MANUAL



BREED'N BETSY TESTIMONIALS

Visit breednbetsy.com.au to listen to students talk about using "Betsy" and how it has helped their progression!

The image shows a screenshot of the website breednbetsy.com.au. At the top, there is a navigation bar with the phone number "Brad: 0428 954 337" and links for "Home", "About Us", "Bovine", and "Equine". The main header features the text "BREED'N BETSY SIMULATOR FOR BOVINE & EQUINE REPRODUCTION TRAINING". Below this, there are two video testimonials. The first is titled "Breed'n Betsy - Matt Testimonial" and shows a man in a green shirt standing next to a simulator. The second is titled "Breed'n Betsy - Darcy Testimonial" and shows a man in a plaid shirt. To the right of the testimonials is a large image of a brown horse and a black and white cow. In the bottom right corner, there is a computer monitor displaying the website's product page, which includes the text "BREED'N BETSY SIMULATOR FOR BOVINE & EQUINE REPRODUCTION TRAINING" and "BREED'N BETSY FRAME SETUP - WATER RECTUM".

**FULL
PRODUCT
CATALOGUE
AVAILABLE
ONLINE**



Scan QR code to watch the videos

www.breednbetsy.com.au

BREED'N BETSY

BOVINE BACK END & PELVIC WALL



Breed'n Betsy is the original and the most highly developed artificial training device on the market, found in classrooms all over the world. This unique training tool was created for teaching and learning the specific skills required to be successful in artificial breeding, pregnancy diagnosis and embryo transfer.

The look, touch and feel approach has proven to be the key to its continued success, with students quickly making links between theory and the hands-on learning provided by the units. With a variety of package options and an unparalleled level of realism and adaptability, Breed'n Betsy has become the 'go to' for universities, high schools and colleges.

From the initial prototypes to the unmatched realism of the device today, I am proud of the continual innovation and development of every element in the Breed'n Betsy product line. I am confident that the products will compliment your teaching and improve student learning outcomes.

If you have any questions or concerns, please don't hesitate to contact me.

Regards,
Brad Pickford


BB01

BOVINE BACK END




WATER RECTUM

BREED'N BETSY BOVINE BACK END FRAME

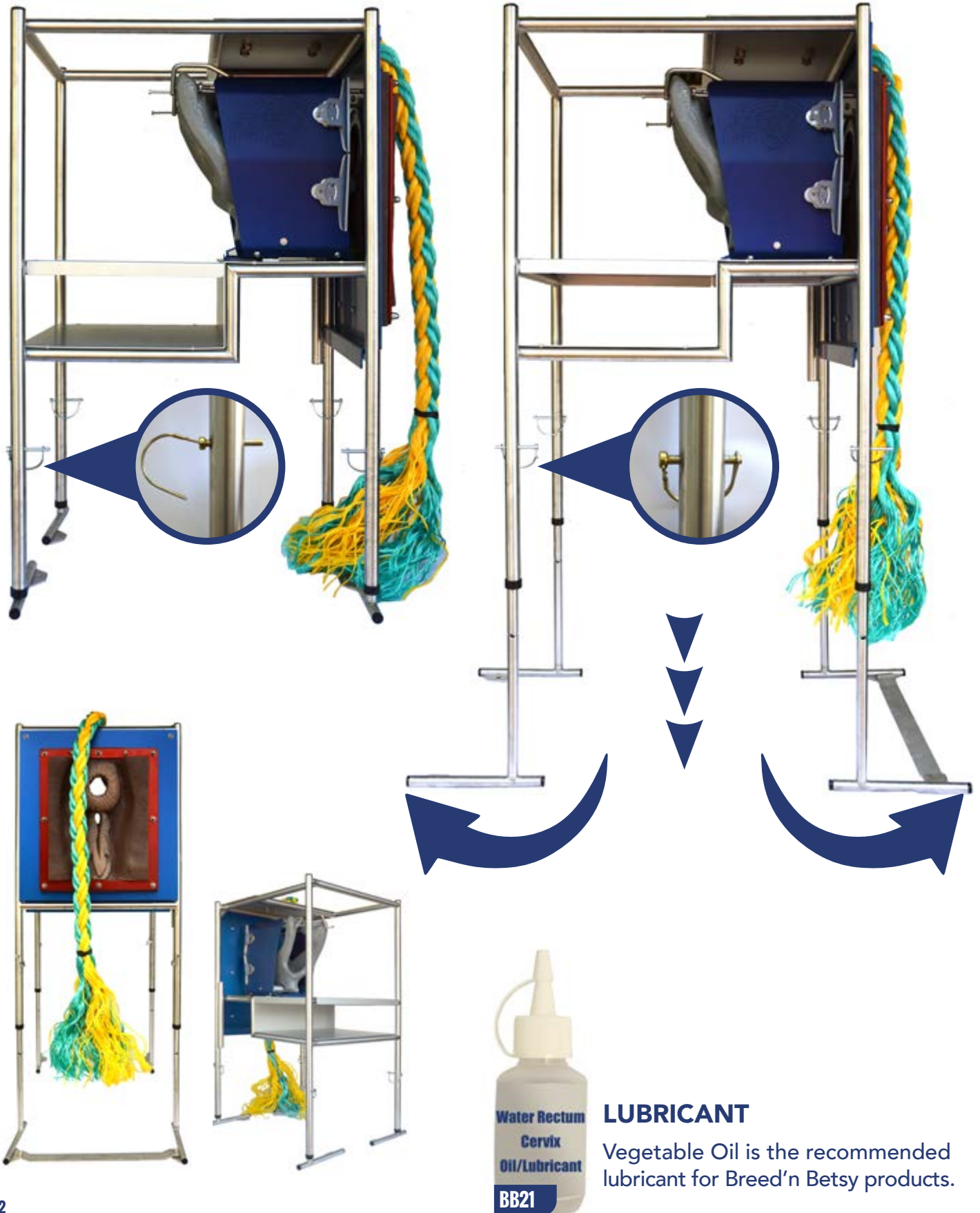

BREED'N BETSY WATER RECTUM (PORTABLE FRAME)

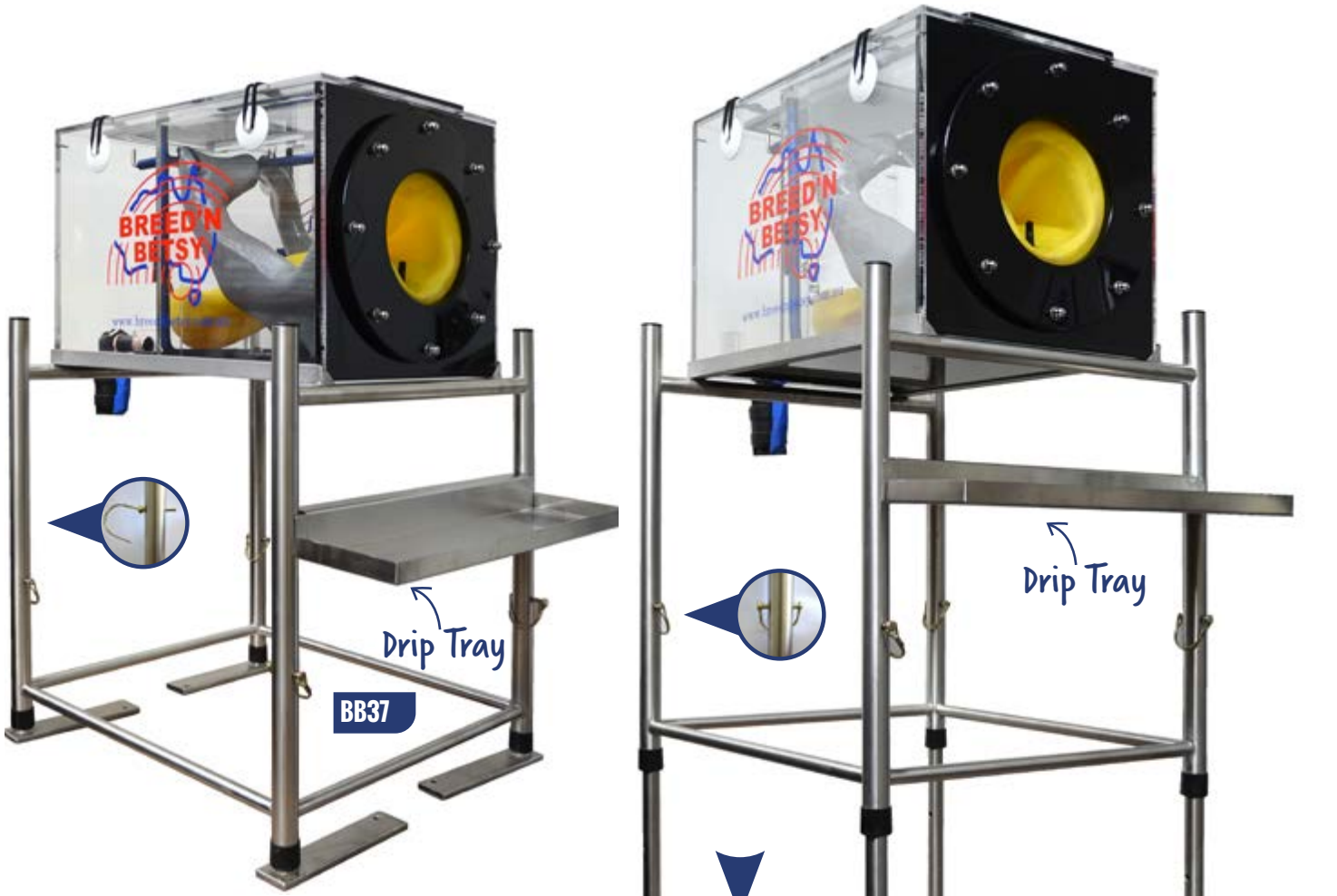

BB11



BB13



BB25



BB23

BB17

Stabilisers

Drip Tray


Drip Tray

BREED'N BETSY PORTABLE FRAME SETUP




SETUP INSTRUCTIONS

1. Fix Top Brace (F) to frame using 4 x Wingnuts (D)
2. Attach Struts (B) to frame with D Clips (C)
3. Extend legs of frame and insert D Clips (C)
4. Position Tray (E) onto centre bar at front of frame.

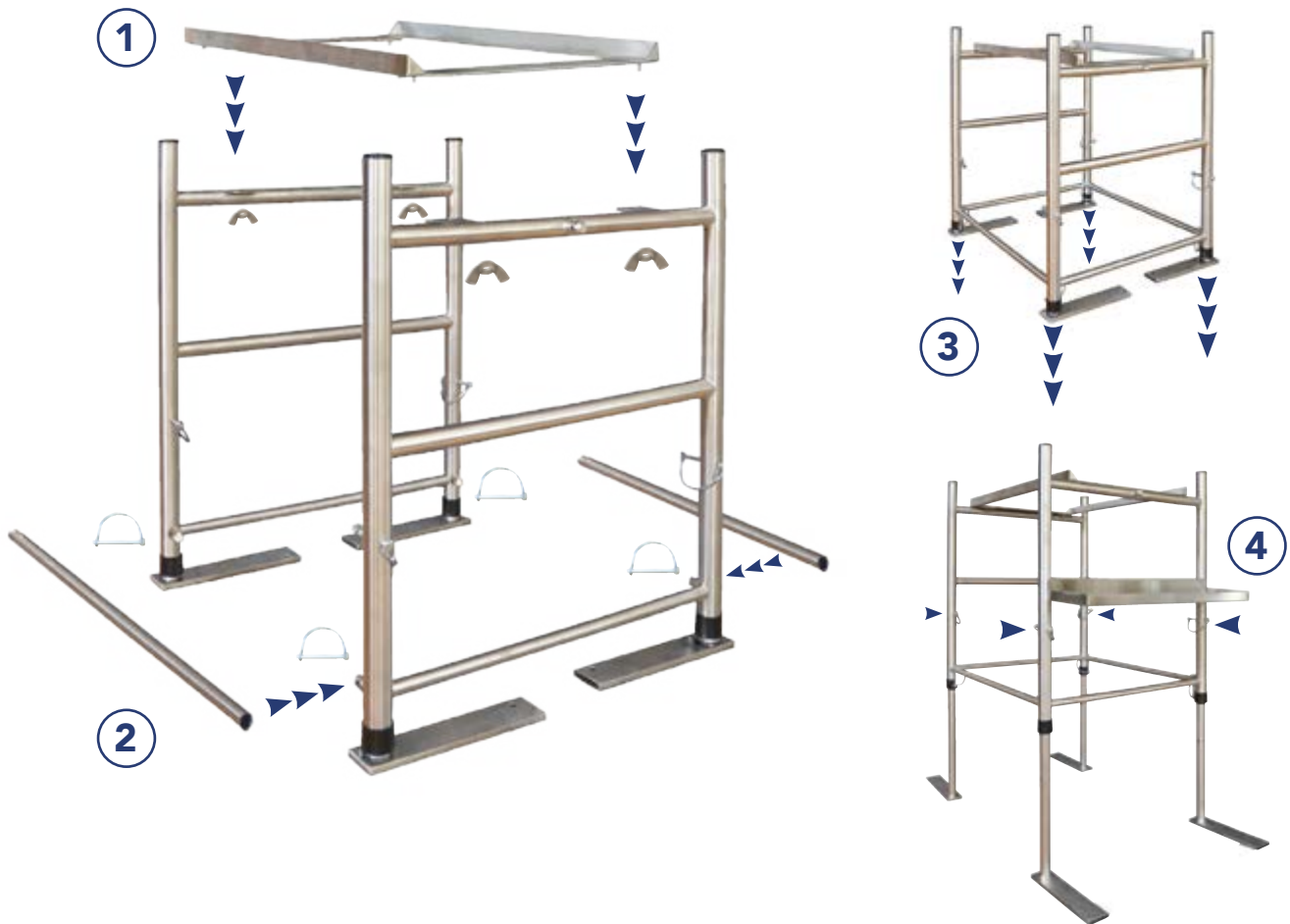

CONTENTS

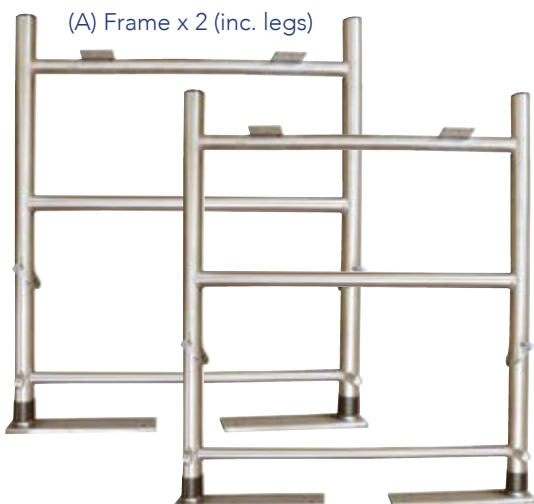


BREED'N BETSY CASTRATING SIMULATOR



SETUP INSTRUCTIONS

1. Fix Calf Unit (G) and Lamb Unit (H) with Wingnuts (M)
2. Attach Calf Tail (I) and Lamb Tail (J)
3. Loop Castrator (K) over frame
4. Slide Tail Skin (L) over vertebrae on Lamb Tail (J)

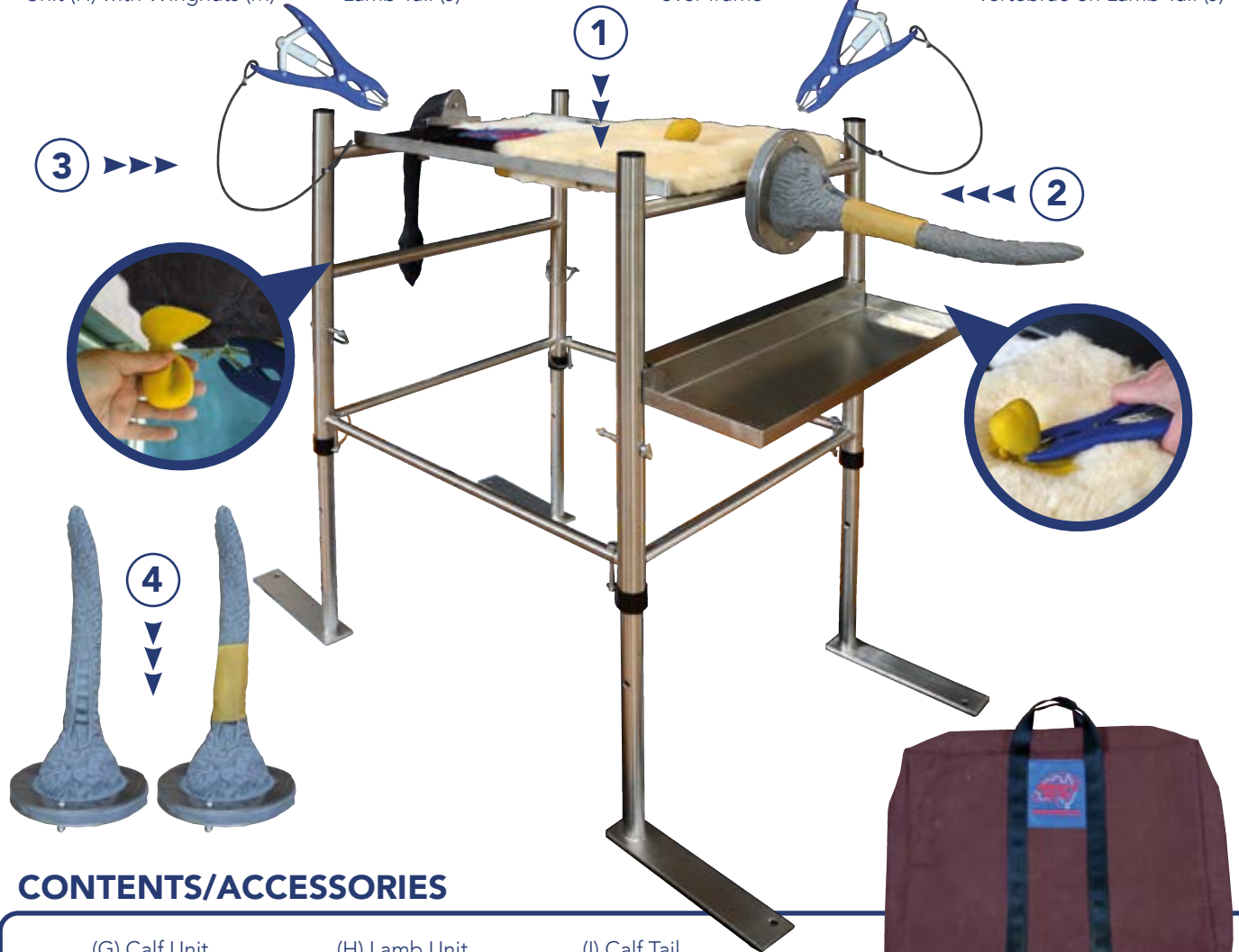

CONTENTS/ACCESSORIES

(G) Calf Unit



(H) Lamb Unit




(I) Calf Tail



(J) Lamb Tail (female)



(L) Tail Skin x 10

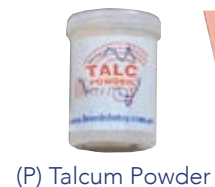

(P) Talcum Powder



(M) Wingnuts x 4



(K) Castration Elastator x 2



(N) Reusable Castrating Rings x 100



(O) Testicles (with cremaster muscle)

BREED'N BETSY BOVINE PELVIC WALL & PELVIS



PELVIC WALL

BB03



BOVINE PELVIS

BB02



PELVIC WALL SETUP



BREED'N BETSY ATTACHMENT SETUP



Clip the broad ligaments to pelvic rods or attachment to pelvic wall to position the ovaries as required. Wash all organs with detergent/disinfectant and allow to dry completely before storage.



BB07



PELVIC ROD CONNECTORS

BB19




PELVIC WALL CLIPS

BB10



USING RECTAL WALL

BB18

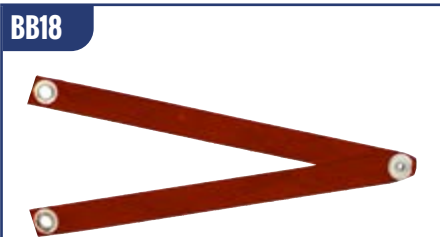

LATEX ORGAN ATTACHMENT

BREED'N BETSY LATEX BREEDING TRACT & CERVIXES



Clip organ attachment to pelvic wall secure organs in the Breed'n Betsy. Wash all organs with detergent/disinfectant and allow to dry completely before storage. Use cervixes individually or in the Latex Breeding Tract.

BB40



LATEX BREEDING TRACT (WITHOUT CERVIX)

BB41



LATEX BREEDING TRACT WITH CERVIX
(CHOICE OF 7 CERVIXES)

BB42



SMALL MAIDEN HEIFER CERVIX (SMALL TEXTURED
ENTRANCE HOLE, 0 CERVICAL FOLDS)

BB43



SMALL CERVIX - OFF-CENTRED ENTRANCE
(1 CERVICAL FOLD)

BB44



SMALL CERVIX WITH ENTRANCE FLAP
(1 CERVICAL FOLD)

BB45



MEDIUM CERVIX WITH ENTRANCE FLAP
(1 CERVICAL FOLD)

BB46



SMALL CERVIX - OFF-CENTRED ENTRANCE
(2 CERVICAL FOLDS)

BB47



SMALL CERVIX WITH ENTRANCE FLAP
(2 CERVICAL FOLD)

BB48





MEDIUM CERVIX WITH ENTRANCE FLAP
(2 CERVICAL FOLD)

BREED'N BETSY COMPLETE BREEDING TRACTS



Clip organ attachment to pelvic wall secure organs in the Breed'n Betsy. Wash all organs with detergent/disinfectant and allow to dry completely before storage.

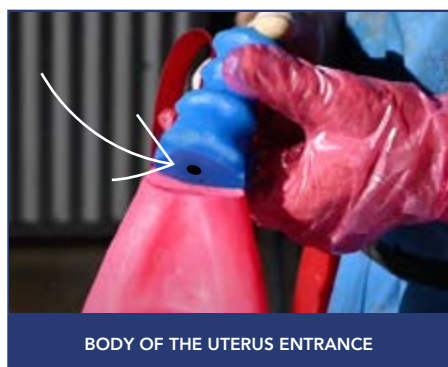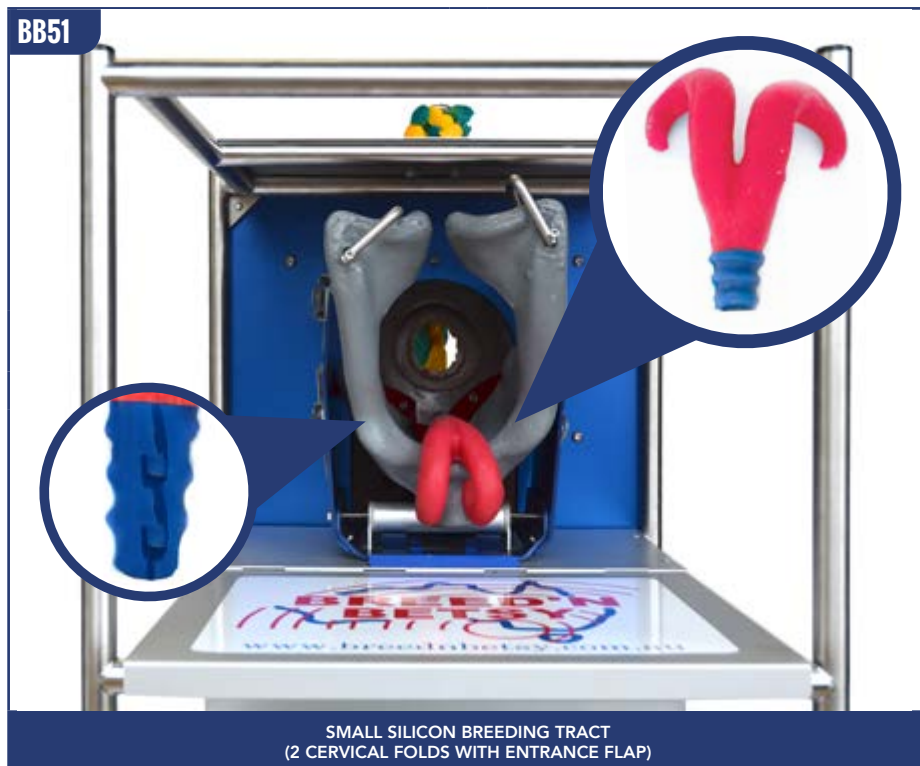

BREED'N BETSY PREGNANCY ORGANS & SETUP



Non Pregnant with ovaries attachments. Clip organ attachment to pelvic wall secure organs in the Breed'n Betsy. Wash all organs with detergent/disinfectant and allow to dry completely before storage. Use Latex Drying Powder to dry inside pregnancy bladders.

BB69



9 WEEK SILICON FOETUS

9 Week Foetus used for visual approach and ultrasound



BB59



6 WEEK PREGNANCY, FLUID FILLED HORN (WITH MEMBRANE ATTACHED INTERNALLY)

BB60



7 WEEK SILICON PREGNANCY

BB61



8 WEEK SILICON PREGNANCY

BB62



9/10 WEEK SILICON PREGNANCY



9/10 WEEK SILICON PREGNANCY



UTERINE ARTERY

BB68



SET OF UTERINE ARTERIES (6 DIFFERENT STAGES OF PREGNANCY - PUNY/4/5/6/7/8MTH)

BREED'N BETSY PREGNANCY ORGANS & SETUP



Clip organ attachment to pelvic wall secure organs in the Breed'n Betsy. Wash all organs with detergent/disinfectant and allow to dry completely before storage. Use Latex Drying Powder to dry inside pregnancy bladders.

BB64




9-11 WEEK LATEX PREGNANCY
WATER BLADDER (WATER FILLED FOR REALISTIC FEEL)

BB66



12 WEEK SILICON PREGNANCY

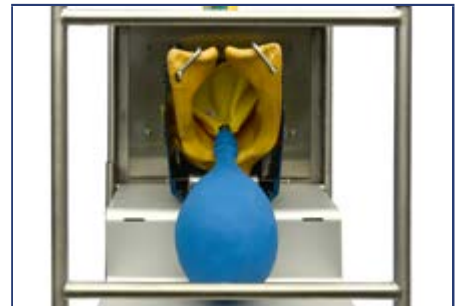

12 WEEK SILICON PREGNANCY

BB63




9 WEEK LATEX PREGNANCY
WATER BLADDER

BB65



12 WEEK PREGNANCY WITH PLACETOMES/
COTYLEGIENS ATTACHED INTERNALLY

BB67

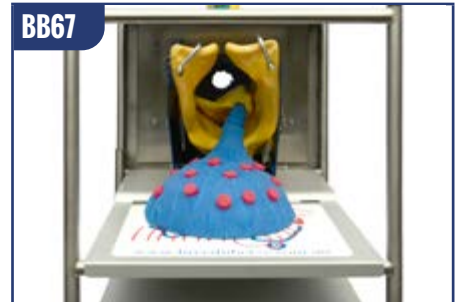

PLACENTA WITH PLACENTOMES



9 WEEK LATEX PREGNANCY BLADDER



11 WEEK LATEX PREGNANCY BLADDER



PLACENTA WITH PLACENTOMES &
UTERINE ARTERIES ATTACHED



BREED'N BETSY WATER RECTUM



WATER RECTUM

Recently updated and now on it's own stand, the Water Rectum is one of the most exciting features of the simulation packages. It provides a unique level of realism for the student - the water creates a floating effect that makes the organs less static requiring more precision and the sphincter places pressure on the user's arm. This combination is unique and allows for continual assessment of student understanding and a higher level of diagnostic training.

Vision from all sides including the top enables trainers to guide students through every step of essential processes including organ/ovary location, cervix location, palpation, retraction/indirect retraction and manipulation of organs.

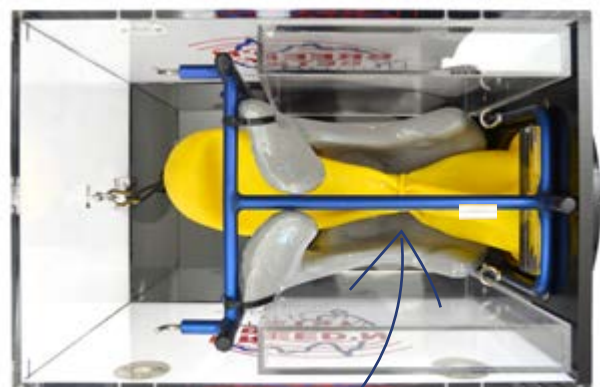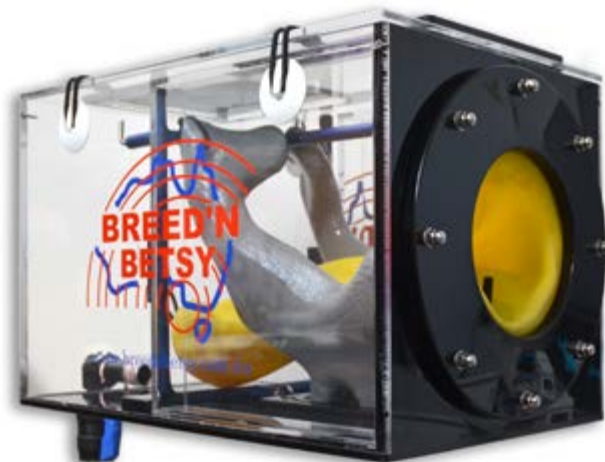

STAINLESS STEEL PIPETTE & VALVE CAPS BB54



BB39



Slide the sphincter ring over the palpation liner



LUBRICANT
Vegetable Oil

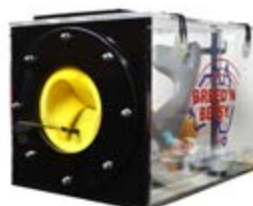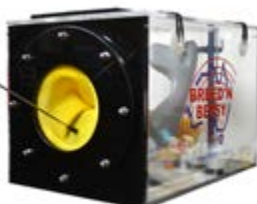

WATER RECTUM USAGE



ARTIFICIAL BREEDING & PREGNANCY DIAGNOSIS



Push valve cap in to seal Water Rectum

ARTIFICIAL BREEDING TRAINING

BB08



WATER RECTUM CARRY BAG

Easy access to internal organs



Palpation Liner



BB38

PALPATION LINER REPLACEMENT



01 ASSEMBLE PARTS



02 FIT SEALING WASHER



03 MAKE SURE WASHER IS CENTRED



04 INSERT PALPATION LINER




05 FLATTEN PALPATION LINER AGAINST WASHER



06 ATTACH COVER



07 TIGHTEN NUTS WITH ALLEN KEY & SPANNER

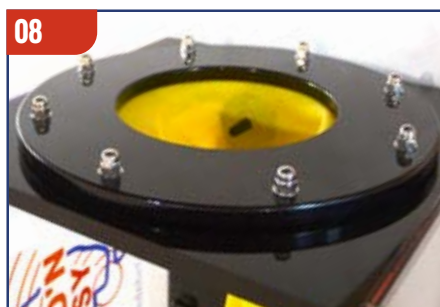

08 COMPLETED COVER ASSEMBLY




09 ATTACH PALPATION LINER CLIP. THE BLUE VIRGINAL FOLD AT THE BOTTOM EXTEND ONTO FRONT WALL

WATER RECTUM BREEDING CERVIX SETUP




Attach the broad ligaments to the hooks on the internal water rectum frame. Use the different eyelet settings to position the cervix closer or further from the back end. Wash all cervixes and water rectum with detergent/disinfectant and allow to dry completely before storage.

BB28

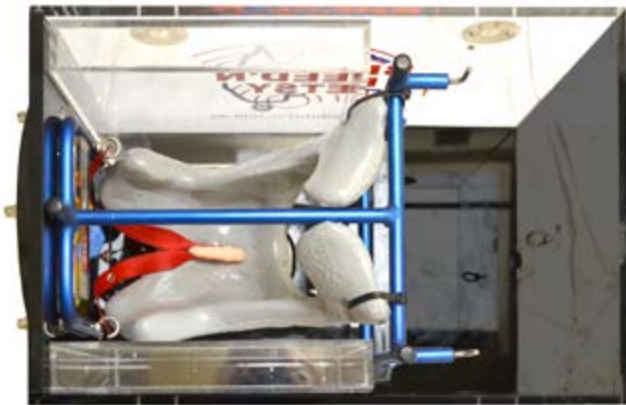


WATER RECTUM SMALL MAIDEN HEIFER CERVIX
(SMALL TEXTURED ENTRANCE HOLE, 0 CERVICAL FOLDS)

BB29

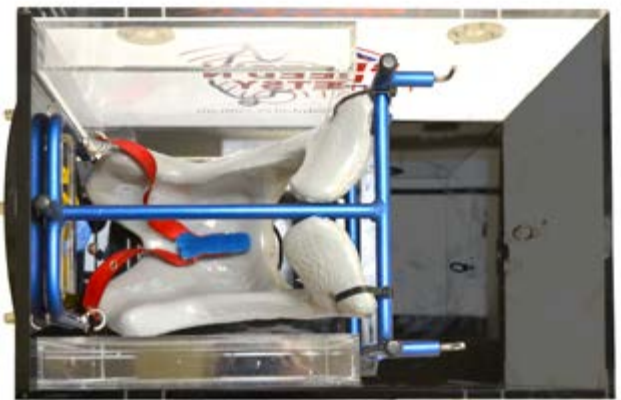


WATER RECTUM SMALL CERVIX - OFF-CENTRED ENTRANCE
(1 CERVICAL FOLD)

BB30

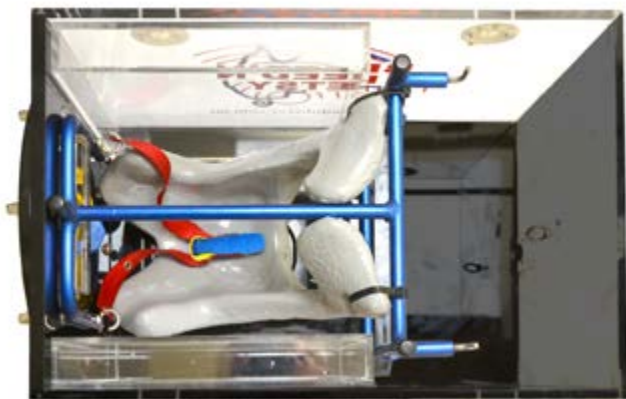

WATER RECTUM SMALL CERVIX WITH ENTRANCE FLAP
(1 CERVICAL FOLD)

BB31

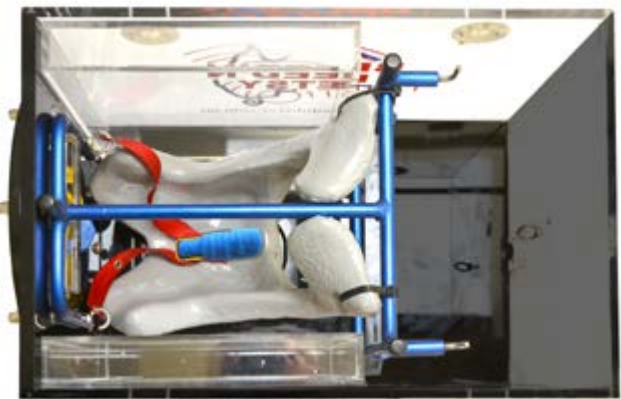

WATER RECTUM MEDIUM CERVIX WITH ENTRANCE FLAP
(1 CERVICAL FOLD)

Breed'n Betsy cervixes offer the user a realistic look, touch and feel!



BB49



BLUE CERVIXES HAVE 1 CERVICAL FOLD

BB50




RED CERVIXES HAVE 2 CERVICAL FOLDS

BB32




WATER RECTUM SMALL CERVIX - OFF-CENTRED ENTRANCE (2 CERVICAL FOLDS)

BB33

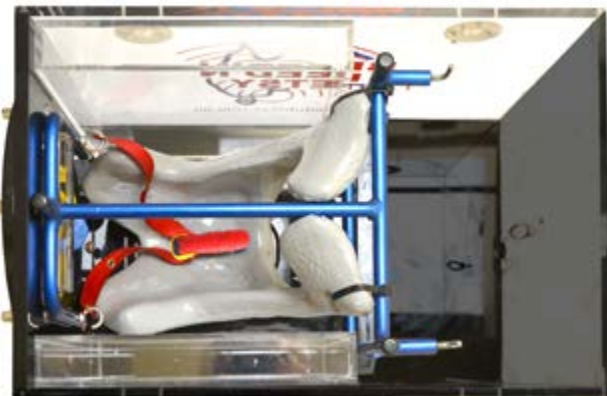

WATER RECTUM SMALL CERVIX WITH ENTRANCE FLAP (2 CERVICAL FOLD)

BB34

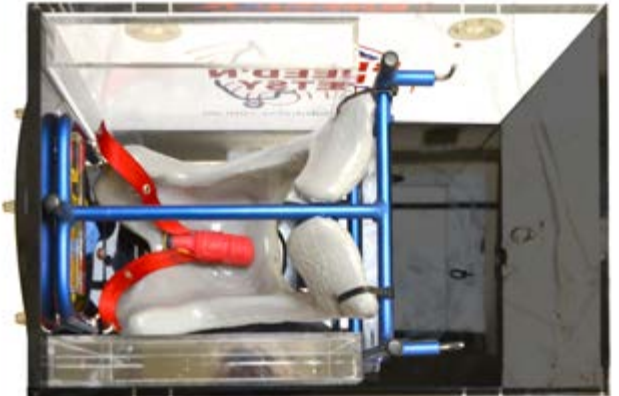


WATER RECTUM MEDIUM CERVIX WITH ENTRANCE FLAP (2 CERVICAL FOLD)

WATER RECTUM INTERNAL ORGAN SETUP



Use the organ attachment to secure organs in the water rectum. Wash all organs and water rectum with detergent/disinfectant and allow to dry completely before storage.

Organ Attachment

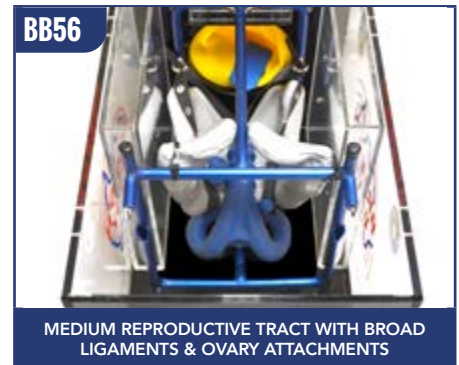



Virginal Ligament



BB52

MEDIUM SILICON BREEDING TRACT
(2 CERVICAL FOLDS WITH ENTRANCE FLAP)



BB53

LARGE SILICON BREEDING TRACT
(2 CERVICAL FOLDS WITH ENTRANCE FLAP)

BB51


SMALL SILICON BREEDING TRACT
(2 CERVICAL FOLDS WITH ENTRANCE FLAP)

BB63



9 WEEK LATEX PREGNANCY BLADDER

BB64

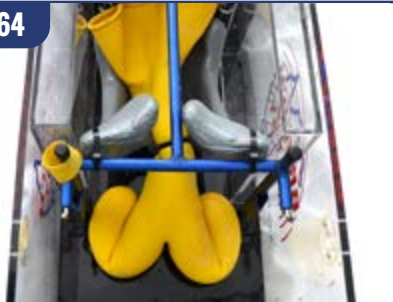


2 HORNS 9-11 WEEK
FILLED LATEX PREGNANCY BLADDER

BB65



11 WEEK LATEX PREGNANCY BLADDER

BB68

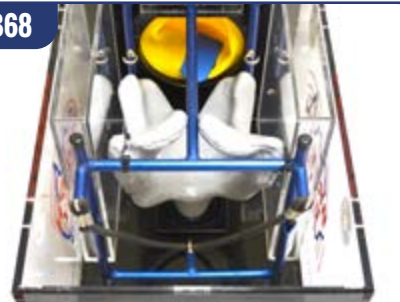

UTERINE ARTERY

BB26



WATER RECTUM ORGAN ATTACHMENT

BB21





LUBRICANT
VEGETABLE OIL

BREED'N BETSY PIPETTE CERVIX LOCATOR



PELVIS ASSEMBLY

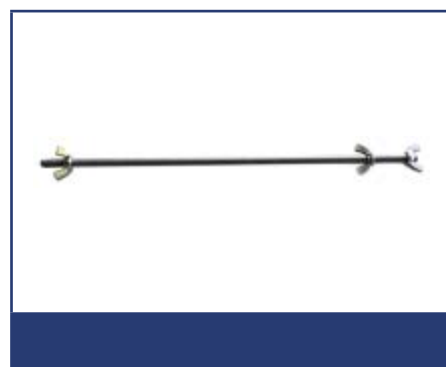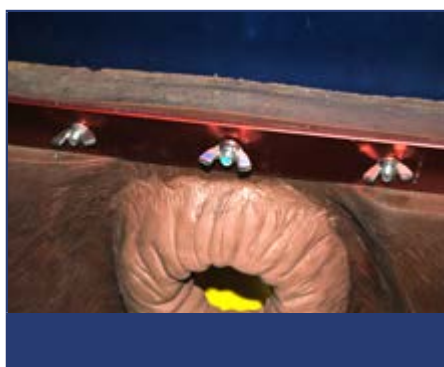

BREED'N BETSY

EMBRYO TRANSFER (BOVINE)



BB70



EMBRYO FLUSHING TRACT

BB71



EMBRYO FLUSHING FLUID
FRAME ATTACHMENT

BB72



EMBRYO IMPLANTING TRACT



BREED'N BONNY PELVIC WALL & BREEDING CERVIX



EQUINE PELVIC WALL & PELVIC FLOOR

BB06




BREEDING CERVIX

Continuous use, realistic equine cervix used for artificial breeding and swab training.



Soft life like cervix with realistic internal and external textures

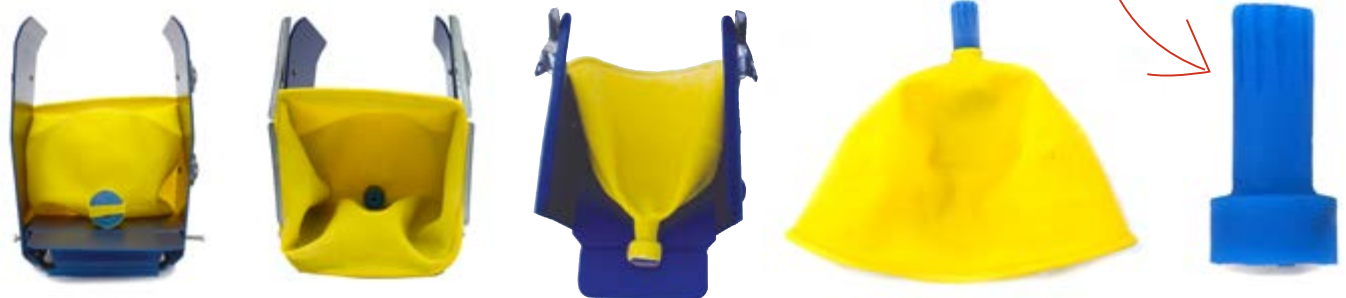

PELVIC FLOOR SETUP



BREED'N BONNY EQUINE BACK END & PELVIS



EQUINE BACK END



EQUINE PELVIS

BB05




BB73



EQUINE BREEDING CERVIX COMPLETE WITH VAGINAL LATEX LIGAMENT ATTACHMENT

BB74

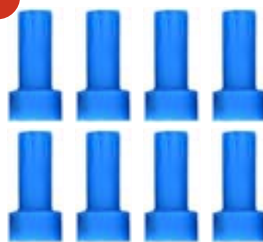

8 X CERVIXES

BB04



EQUINE BACK END



INSEMINATING THE CERVIX FROM BONNIE'S BACK END



TRAINING VETERINARY STUDENTS AT CHARLES STURT UNIVERSITY




EQUINE BACK END

BREED'N BONNY TRACTS, PREGNANCIES & OVARIES



Bonny is a realistic, year round training device for students to acquire skills in palpation and retraction in the equine reproductive cycle.

REPRODUCTIVE TRACTS

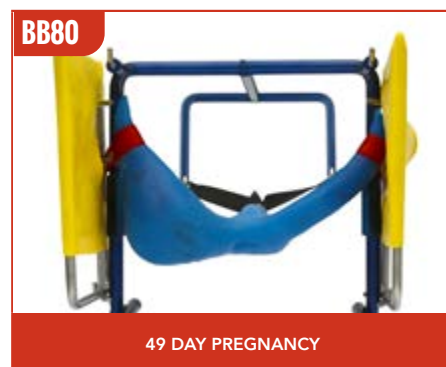




BB81



56 DAY PREGNANCY (PALPATION)

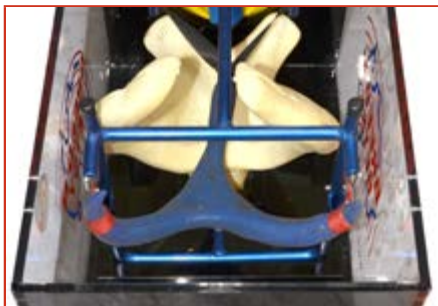


BREEDING TRACT SETUP

BB81



56 DAY PREGNANCY (PALPATION)

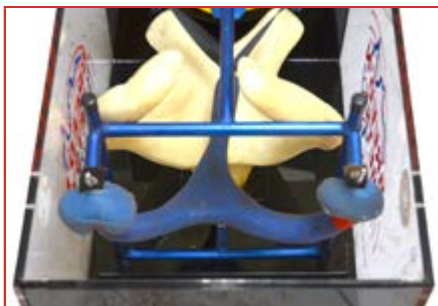

BREEDING TRACT & OVARY SETUP

BB09



ACCESSORIES CARRY BAG

BB82



INTERCHANGEABLE OVARIES WITH FOLLICLES

BB26



WATER RECTUM ORGAN ATTACHMENT

BREED'N BONNY EQUINE FIXED FRAME



PELVIS ASSEMBLY





**BREED'N
BETSY**

www.breednbetsy.com.au



*Better
Better
Better*



EQUINE

COLORADO LATEX WATER JACKET STRAIGHT



EQUINE

COLORADO LATEX REUSE FLARED/TAPERED



EQUINE

FHK LATEX WATER JACKET TAPERED



EQUINE

HANOVER LATEX WATER JACKET FLARED



EQUINE

MISSOURI TYPE AV LINER

Thicker, more durable latex!

Feel Sensation Performance



EQUINE
MISSOURI TYPE AV LINER



FACTORY TO YOU!
PERFECT
FACTORY TO YOU!

BREED'N BETSY BOVINE PACKAGES



No.	BOVINE PACKAGES	USAGE
01	COMPLETE BOVINE PACKAGE This package contains one Water Rectum and Stand along with one Bovine Back End and Frame. Complete Silicone Breeding Tracts, Open Tracts, Latex Breeding Tracts, Ovaries, Water Rectum Cervixes and Cervixes. Two training units that can be used for Artificial Breeding, Pregnancy Diagnosis, Ovary Palpation, Open Reproductive Tracts Palpation & Retraction.	FF WR OV OT LT C WC BT PD
02	COMPLETE WATER RECTUM PACKAGE This package contains one Water Rectum and Stand. Complete Silicone Breeding Tracts, Ovaries, Water Rectum Cervixes and Cervixes. It is a flexible training option with the Water Rectum providing appropriate simulation for students of all abilities and needs. Suitable for Artificial Breeding, Pregnancy Diagnosis, Ovary Palpation, Open Reproductive Tracts Palpation & Retraction.	FF WR OV OT LT C WC BT PD
03	COMPLETE BOVINE BACK END PACKAGE This package contains one Bovine Back End and Frame. Complete Silicone Breeding Tracts, Open Tracts, Latex Breeding Tracts and Cervixes. It is a good option for smaller teaching classes and introductory to Bovine Reproduction. Suitable for Artificial Breeding, Pregnancy Diagnosis and Open Reproductive Tract Palpation & Retraction.	FF WR OV OT LT C WC BT PD
04	ARTIFICIAL BREEDING PACKAGE This package contains one Water Rectum and Stand along with one Bovine Back End and Frame. Open Tracts, Ovaries, Water Rectum Cervixes and Cervixes. Suitable for Artificial Breeding with two training units that will introduce your students to Artificial Insemination and Palpation of empty tracts. This package has both units allowing you to have multiple stations set up for teaching.	FF WR OV OT LT C WC BT PD
05	ARTIFICIAL BREEDING PACKAGE (WATER RECTUM ONLY) This package contains the Water Rectum and Stand. Suitable for Artificial Breeding it also includes Open Tracts, Ovaries, Water Rectum Cervixes and Cervixes designed specifically for the use in the Water Rectum.	FF WR OV OT LT C WC BT PD
06	ARTIFICIAL BREEDING PACKAGE (BOVINE BACK END AND FRAME ONLY) This package contains one Bovine Back End and Frame. Suitable for Artificial Breeding this package is a good beginner set for an introduction into Bovine Artificial Insemination and includes Complete Silicone Breeding Tracts, Open Tracts, Ovaries and Cervixes.	FF WR OV OT LT C WC BT PD
07	WATER RECTUM This package includes the Water Rectum and Stand which can be added onto one of the other packages if you are looking for an additional work station. Does not include any internal organs.	FF WR OV OT LT C WC BT PD
08	BOVINE BACK END & FRAME This package contains a Bovine Back End and Frame that can be added onto one of the other packages if you are looking for an additional work station. Does not include any internal organs.	FF WR OV OT LT C WC BT PD

KEY

- FF** Fixed Frame **WR** Water Rectum **OV** Ovaries **BT** Complete Silicone Breeding Tracts **OT** Open Tracts
LT Latex Breeding Tracts **C** Cervixes **WC** Water Rectum Cervixes **PD** Pregnancy Diagnosis

BREED'N BONNY EQUINE PACKAGES



No.	PACKAGE	
09	<p>COMPLETE EQUINE PACKAGE</p> <p>This package contains one Water Rectum and Stand along with one Equine Back End and Frame. Open Tract, Latex Breeding Tracts, Ovaries, Pregnancies and Cervixes. Two training units set up for Equine Pregnancy Diagnosis, Open Reproductive Tract Palpation & Retraction, Ovary Palpation and Artificial Breeding/Swab Collection training.</p>	
10	<p>EQUINE ARTIFICIAL BREEDING PACKAGE</p> <p>This package contains one Equine Back End and Frame. Latex Breeding Tracts and Cervixes. This package can be used for Artificial Breeding and Swab Collection Training.</p>	
11	<p>EQUINE BACK END AND FRAME</p> <p>This package can be added to any existing frame to advance training with in the areas of Artificial Breeding and Swab Training. Does not include any internal organs.</p>	
12	<p>EQUINE/BOVINE WATER RECTUM</p> <p>This package can be added to any existing frame to advance training with in areas of Pregnancy Diagnosis and Ovary Palpation. Does not include any internal organs.</p>	

COMBO PACKAGES



No.	PACKAGE	
13	<p>COMPLETE BOVINE/EQUINE PACKAGE</p> <p>This package contains one Water Rectum and Stand along with one Equine Back End and Frame. Bovine Pelvis and Bovine/Equine Pelvic Wall. Complete Silicone Breeding Tracts, Open Tracts, Latex Breeding Tracts, Ovaries, Water Rectum Cervixes and Cervixes. Two training units that can be used for Artificial Breeding, Pregnancy Diagnosis, Ovary Palpation, Open Reproductive Tracts Palpation & Retraction. Does not include any equine internal organs, refer to product list.</p>	

CASTRATING PACKAGES



No.	PACKAGE	
14	<p>COMPLETE CASTRATING PACKAGE</p> <p>This package contains Portable Frame, Canvas Bag on wheels, Calf Unit, Lamb Unit, Calf Tail, Lamb Tail (female), Wingnuts x 4, Castration Elastrator x 2, Reusable Castrating Rings x 100, Tail Skin x 10, Talcum Powder, Testicles (with cremaster muscle)</p>	
15	<p>CASTRATING PACKAGE</p> <p>This package contains Calf Unit, Lamb Unit, Calf Tail, Lamb Tail (female), Wingnuts x 4, Castration Elastrator x 2, Reusable Castrating Rings x 100, Tail Skin x 10, Talcum Powder, Testicles (with cremaster muscle)</p>	

BREED'N BETSY PRODUCT LIST



CODE	PRODUCT	PAGE
BREED'N BETSY BOVINE / EQUINE FRAME ACCESSORIES		
BB01	1 x Betsy Bovine Back End	1
BB02	1 x Betsy Bovine Pelvis	4
BB03	1 x Betsy Pelvic Wall	4
BB04	1 x Equine Back End	17
BB05	1 x Equine Pelvis	17
BB06	1 x Equine Pelvic Wall	16
BB07	1 x Set of Pelvic Rod Connectors	5
BB08	1 x Water Rectum Carry Bag	10
BB09	1 x Accessories Carry Bag	19
BB10	1 x Latex Rectal Wall	3
BB11	4 x Plastic Frame Leg Plugs	
BB12	4 x Plastic Frame Plugs	
BB13	4 x Frame Leg Pins (Stainless Steel Frame / Fixed Frame)	3
BB14	10 x Frame Leg Pins (Aluminum Portable Frame)	
BB15	4 x Plastic Plugs with hand screw threads for frame attachment (Portable Frame only)	
BB16	8 x Hand Screws (Portable Frame only)	
BB17	1 x Frame Stabilisers	3
BB18	1 x Latex Organ Attachment	5
BB19	4 x Pelvic Wall Clips (with bolts and screws)	5
BB20	1 x Latex Drying Powder (Latex Pregnancy Water Bladders)	3
BB21	1 x Cervix Lubricant Oil	2,15
BB22	1 x Set of Latex Broad Ligaments (Equine)	
BB23	1 x Latex Repair Glue	3
BB24	1 x Water Rectum Acrylic Box Repair Kit	
BB25	1 x Silicon Repair Glue	3
BB26	1 x Water Rectum Organ Attachment	15
BB27	1 x Water Rectum Water Tap Attachment	
BB37	1 x Water Rectum Drip Tray	3
BREED'N BETSY BOVINE WATER RECTUM ACCESSORIES		
BB28	1 x Water Rectum Small Maiden Heifer Cervix (Small textured entrance hole, 0 cervical folds)	12
BB29	1 x Water Rectum Small Cervix - Off-centred entrance (1 cervical fold)	12
BB30	1 x Water Rectum Small Cervix with entrance flap (1 cervical fold)	12
BB31	1 x Water Rectum Medium Cervix with entrance flap (1 cervical fold)	12
BB32	1 x Water Rectum Small Cervix - Off-centred entrance (2 cervical folds)	13
BB33	1 x Water Rectum Small Cervix with entrance flap (2 cervical fold)	13
BB34	1 x Water Rectum Medium Cervix with entrance flap (2 cervical fold)	13
BB51	1 x Small Silicon Breeding Tract (1-3 year old cow) (2 cervical folds with entrance flap)	6,15
BB52	1 x Medium Silicon Breeding Tract (3-5 year old cow) (2 cervical folds with entrance flap)	6,15
BB53	1 x Large Silicon Breeding Tract (5+ year old cow) (2 cervical folds with entrance flap)	6,15
BB38	2 x Latex Palpation Liners (with new attachment for Breeding Valve, Breeding Tracts and Cervixes)	11
BB39	12 x Anus Sphincter Rings	10

CODE	PRODUCT	PAGE
BREED'N BETSY ARTIFICIAL BREEDING ACCESSORIES		
BB40	1 x Latex Breeding Tract without Cervix	6
BB41	1 x Latex Breeding Tract with Cervix (choice of 7 Cervixes)	6
BB42	1 x Small Maiden Heifer Cervix (Small textured entrance hole, 0 cervical folds)	6
BB43	1 x Small Cervix - Off-centred entrance (1 cervical fold)	6
BB44	1 x Small Cervix with entrance flap (1 cervical fold)	6
BB45	1 x Medium Cervix with entrance flap (1 cervical fold)	6
BB46	1 x Small Cervix - Off-centred entrance (2 cervical folds)	6
BB47	1 x Small Cervix with entrance flap (2 cervical fold)	6
BB48	1 x Medium Cervix with entrance flap (2 cervical fold)	6
BB49	1 x Small Cervix cut open (1 cervical fold)	13
BB50	1 x Small Cervix cut open (2 cervical folds)	13
BB51	1 x Small Silicon Breeding Tract (1-3 year old cow) (2 cervical folds with entrance flap)	6,15
BB52	1 x Medium Silicon Breeding Tract (3-5 year old cow) (2 cervical folds with entrance flap)	6,15
BB53	1 x Large Silicon Breeding Tract (5+ year old cow) (2 cervical folds with entrance flap)	6,15
BB54	6 x Stainless Steel Pipette Breeding Rods with Water Rectum Valve Caps	10

BREED'N BETSY BOVINE NON-PREGNANT REPRODUCTIVE TRACTS AND OVARIES		
BB55	1 x Small Silicon Reproductive Tract (heifer size) with Ovaries attached	6,14
BB56	1 x Medium Silicon Reproductive Tract (2-3 year old cow) with Broad Ligaments to attach Ovaries	6,14
BB57	1 x Large Silicon Reproductive Tract (5+ year old cow) with Broad Ligaments to attach Ovaries	6,14
BB58	11 x Interchangeable Ovaries (including: Non Cycle, Corpus Luteum, Cyst and Follicles)	6

BREED'N BETSY BOVINE PREGNANCY DIAGNOSIS		
BB59	1 x 6 Week Pregnancy, Fluid Filled Horn (with membrane attachment)	8,14
BB60	1 x 7 Week Silicon Pregnancy	8,14
BB61	1 x 8 Week Silicon Pregnancy	8,14
BB62	1 x 9/10 Week Silicon Pregnancy	8,14
BB63	1 x 9 Week Latex Pregnancy Water Bladder	9,15
BB64	1 x 9-11 Week Latex Pregnancy Water Bladder (2 horns)	9,15
BB65	1 x 11 Week Latex Pregnancy Water Bladder	9,15
BB66	1 x 12 Week Silicon Pregnancy	9,14
BB67	1 x Silicon Placenta with 4 month placentomes	9,14
BB68	1 x Set of Uterine Arteries (5 different stages of pregnancy - Punny/5/6/7/8mth)	8,15
BB69	1 x 9 Week Silicon Foetus	8

EMBRYO TRANSFER (BOVINE)		
BB70	1 x Embryo Flushing Tract	19
BB71	1 x Embryo Flushing Fluid Frame Attachment	19
BB72	1 x Embryo Implanting Tract	19

BREED'N BONNY EQUINE ACCESSORIES		
BB73	1 x Equine Breeding Cervix complete with Vaginal Latex Ligament attachment	17
BB74	8 x Cervixes	17
BB75	1 x Non Pregnant Reproductive Tract	18
BB76	1 x 16 Day Pregnancy Internal Fluid with Vesicles	18
BB77	1 x 28 Day Pregnancy	18
BB78	1 x 35 Day Pregnancy	18
BB79	1 x 42 Day Pregnancy	18
BB80	1 x 49 Day Pregnancy	18
BB81	1 x 56 Day Pregnancy	19
BB82	10 x Ovaries with Follicles	19

HINTS FOR SUCCESSFUL OPERATION OF "BREED'N BETSY"


1. Apply sufficient lubricant both internally and externally, to the anus, to allow free hand and arm movement.
2. Use artificial breeding gloves (lubricated) with every insertion.
3. A sheath should be used when using normal insemination guns loaded with non-active .25cc (mini) or .50cc (maxi) straws.
4. If cervix is dry or sticking to the insemination gun, apply sufficient lubricant to allow the insemination gun to slide freely.
5. After use, always wash Betsy down with detergent/disinfectant and water to remove all debris.
6. After water rectum use wash everything (frame, water rectum box, organs, liner) with detergent/disinfectant then allow to dry. Reverse out palpation liner to dry. Make sure all latex pieces are dry before storage.
7. Water Bladders: blow half up with air and close the tap this will stop the latex sticking together when dried.

BREED'N BETSY

THE REAL FEEL!

PHOTOS



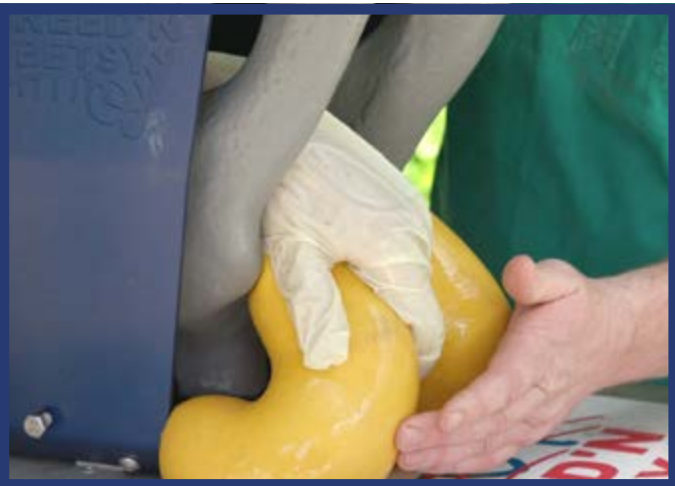



BREED'N BETSY

THE REAL FEEL!



**Locating the correct vertebrae/position for tail docking.*



**The retractable testicle moves up and down the body cavity and scrotum so the students can palpate the testicles into the correct position.*



Brad Pickford
brad@breednbetsy.com.au
Mobile: 0428 954 337
www.breednbetsy.com.au